

APPLICATION CHART

FOR THE RIGHT CHOICE IN COMPRESSION THERAPY ...

FACIAL SUPPORT

Sizes Applicable:

E, F



Head Band



(With Ear Holes)

UPPER EXTREMITY

Sizes Applicable:

Small	A, B
Medium	B, C
Large	D



Gauntlet to Wrist



Gauntlet to Elbow



Gauntlet to Axilla



Elbow Support (Double Layered)

LOWER EXTREMITY

Sizes Applicable:

Small	A, B, C
Medium	C, D
Large	D, E, F, G
Full Leg	S, M , L



Below Knee



Full Leg (Shaped Support Covers) (Double Layered)



Knee Cap



Thigh Covers



Ankle Support (Double Layered)

TORSO

Sizes Applicable :

	19.19
Small	E, F
Medium	G, H
Large	H, I

Hip spica - Prevent haemotoma & odema on operative sites. Breast care support - Mammoplasty & Post Mastectomy.



Torso Cover



(Double Layered)



Breast Care Support Abdominal Support (Double Layered)



Hip Spica

IMPORTANT:

- Choose correct size by referring to Calibration Chart. Incorrect size may lead to swellings
- · Never overlap the covers from upper end.
- For additional pressure (if required) in each size, two layers of Grippon Tubular Support cover may be used, one rolled over the other.

ORDERING DETAILS:

• Various size are available in I/4, I/2, 3/4 & I m. length. Also available in professional packing of 5m. & 10 m rolls.

• Never cut though the elastic of Grippon cover or stitch.

For further details, please contact:

TECHNOMED INDIA PVT. LTD.

Regd. Office: H-107, Shivaji Park, Punjabi Bagh, New Delhi-110026 01141420555 (+91 8586952121

E-mail: info@technomed.net | technomedindia@outlook.com Website: www.technomed.net | www.pressuregarment.com

CALIBRATION CHART

SIZE	CIRCUMFERANCE AT WIDEST POINT IN CENTIMETERS
Size A	14 to 22 cm
Size B	20 to 30 cm
Size C	28 to 40 cm
Size D	36 to 50 cm
Size E	45 to 56 cm
Size F	51 to 62 cm
Size G	57 to 70 cm
Size H	69 to 85 cm
Size I	80 to 120 cm

FOR THE RIGHT CHOICE IN COMPRESSION THERAPY



ADVANTAGE:

- ✓ A convenient substitute to Crepe Bandage, no need to tie, just pull like a socks.
- ✓ Full flexibility in movement, while covers stay firm in place without pins or tapes.
- Controlled & consistent compression, provide comfortable and effective support
- ✓ Light and comfortable to wear.
- ✓ Porous structure permits skin to breathe.
- ✓ Washable and reusable.
- ✓ Full range of sizes to suit any application.
- ✓ Easy to apply & reapply.
- ✓ Variable pressure gradient possible by using double or more layer of the covers.
- ✓ Easily adapts to body contour.
- Seamless construction avoid pressure points.
- ✓ Seamed end increses life of covers.
- ✓ Light enough to be worn beneath normal wear & shoes.
- ✓ Reversible for use on either side.
- ✓ Retains body heat to ease pain.

ELASTICATED TUBULAR SUPPORT COVERS

INDICATIONS:

- ✓ Treatment of sprains, strains, swelling of muscular & join problems due to work and sports injuries or dislocations.
- Rehabilitation of weakining muscles following orthopaedic injuries & long duration plaster casts.
- ✓ Reduces oedema's, relieves swelling & assist venous circulation i.e. varicose veins, deep vein thrombosis, stump support.
- ✓ Treatment of hypertrophy scar following burns or deep dermal injury i.e. burn scar hypertrophy, post operative hypertrophy.
- ✓ Prevent the formation of varicose veins in pregnancy & gives most comfortable support in established conditions.
- ✓ Keeps limb warm & comfortable after frictions & massages.
- ✓ Skin donor sites & skin grafted areas.
- ✓ Lumbar/ Abdominal Support i.e. post pregnancy abdominal binder.
- ✓ Breast care dressing i.e. post mastectomy & mammoplasty support.
- ✓ Chest Binder i.e. following rib injuries, post thoracic surgery support
- ✓ Hip Spica i.e. hip injury, arthritis of hip joints.
- ✓ Prevention of decubitus ulcers of elbow & heels, pads may be placed under Grippon covers, next to the skin.
- Recommended for the support of amoutation stump.



TECHNOMED INDIA PVT. LTD.

E-mail: info@technomed.net | technomedindia@outlook.com Website: www.technomed.net | www.pressuregarment.com